GW07: Biochemical Identification of Microorganisms (API System)

Before Starting Always determine:

- Cell shape: cocci or bacilli
- Gram stain: Gram (+) or Gram (-)


Then:

- Gram (-) bacilli → **oxidase test**
- Gram (+) cocci → **catalase test**

Oxidase and Catalase Tests (Simple Explanation)

The **oxidase test** shows if bacteria use oxygen in respiration using a specific enzyme. You place the colony on a disk.

- Purple color → bacteria use this enzyme (**positive**)
- No color → (**negative**)
- * Enterobacteria = oxidase (-)

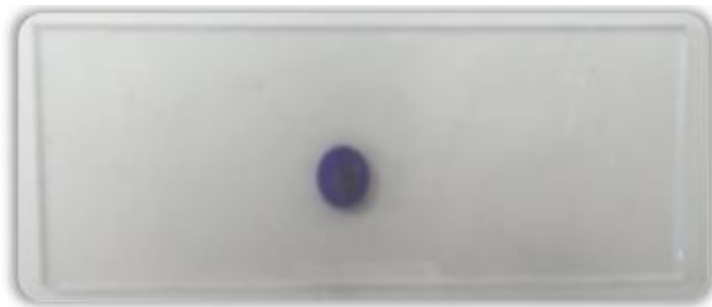Oxydase +

The **catalase test** shows if bacteria can protect themselves from toxic oxygen products. You add hydrogen peroxide.

- Bubbles → oxygen is released → (**positive**)
- No bubbles → (**negative**)
- Staphylococcus (+) | Streptococcus (-)

Catalase +

API System (What students must understand)

API is a **set of small biochemical tests in one strip**. Each small tube tests a bacterial activity (enzyme, sugar use, etc.). During incubation: Bacteria grow, They change the color of the tests This gives a **biochemical profile**

A strip with **20 small tubes (tests)**. Each tube contains a **specific substrate**. Color changes = bacterial activity

Example of tests inside:

- Sugar fermentation (GLU, MAN...)
- Enzymes (URE, GEL...)
- Metabolism (CIT, H₂S...)

Galerie API20E

Procedure

1. Prepare the strip (humid box)
2. Prepare bacterial suspension (0.5 McFarland)
3. Fill all tubes with bacteria

Some tubes need **no oxygen** → we add oil. This allows bacteria that grow without oxygen to react

4. Incubate:
 - **37°C for 18–24 h**

□ Reading Results

After incubation: Two types of reactions:

1. Direct color change



- Example: sugar fermentation → yellow color
- Means bacteria **use that sugar**

2. Hidden reaction (needs indicator)

- Some bacteria produce invisible products
- We add a chemical to **reveal the reaction**

Example:

- IND test → shows if bacteria break down tryptophan
- VP test → shows fermentation type

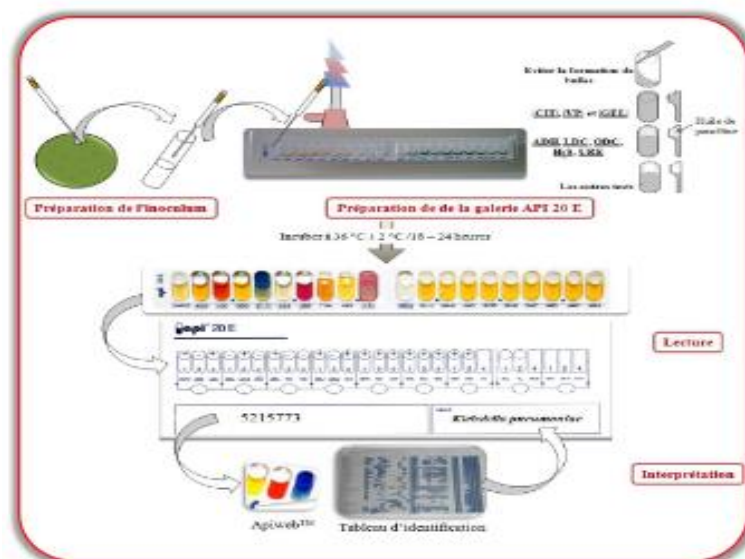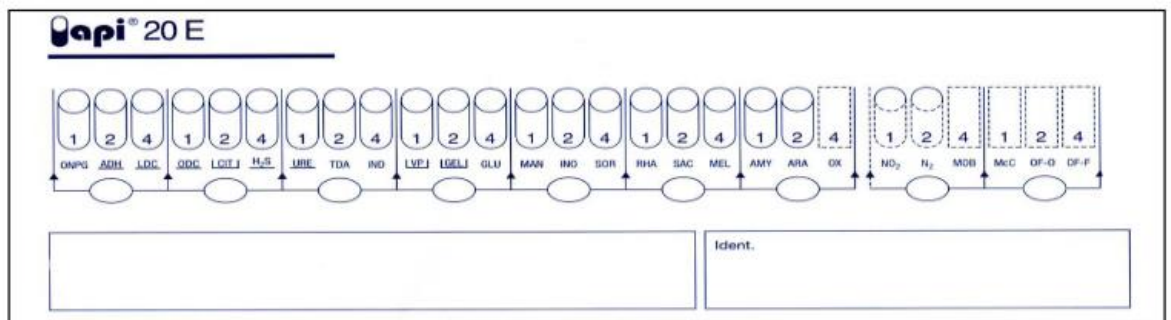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